




VX 610

Quick start

STEP 1: BEFORE SWITCHING ON THE DEVICE

First login

- Go to the the web site <https://nexus.visionix.com/login>
Insert the credentials that you received in your mail:
 - USERNAME:** insert the username of your account
 - PASSWORD:** insert the temporary password of your account.

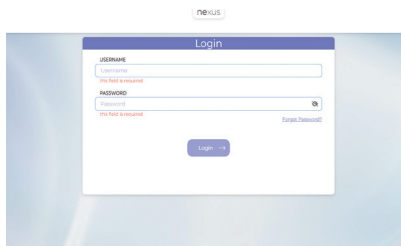

New password

- Choose a new password and click on "Continue"

FIRST ACCESS - ACTIVATION

Insert a new password to get access to the account

NEW PASSWORD

new password

CONFIRM NEW PASSWORD

confirm new password


Continue

Password must contain:

- 8 chars min
- At least 1 uppercase char
- One number or one special char

PUK code


- Download and **keep the PUK code, it's mandatory for account recovery.** Insert the PUK code just downloaded and then click "Continue".

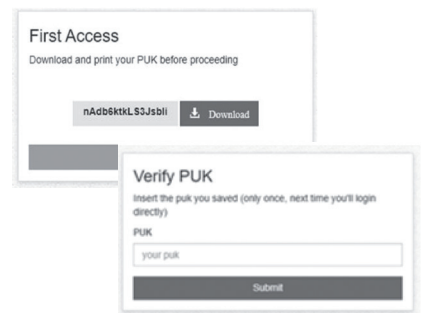

STEP 2


Switch on :

- The device by pressing the green button at the back for 3 seconds.
- The tablet, click on the vx 610 logo and use the new password that you've created on Nexus.

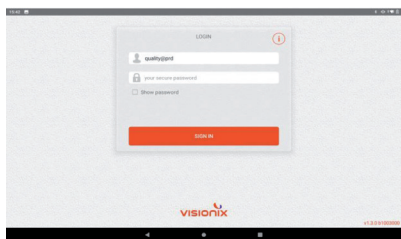


STEP 3: HOW TO START AN EXAM


- Create a new patient**
 - a/ Patients > New Patient > Create
 - or
 - b/ Do exam > Enter new patient > Create

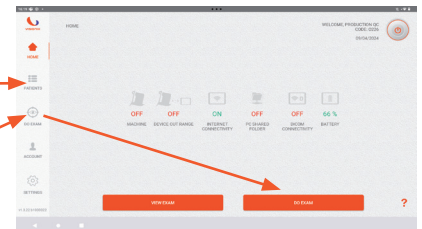

- Complete all the patient information**

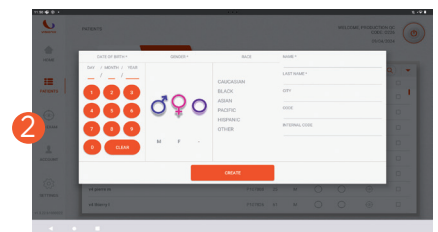

- Select the type of fixation that you want on the left and/or right eye.
 - Press Start to launch the exam.

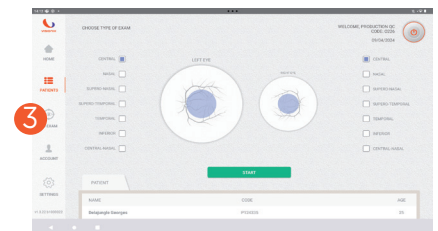

Quick start




For more detailed instructions and special features please refer to the specific user manuals on <https://academy.visionix.com/>


STEP 4: ANALYSIS

- The exam is done.
- You can see the fundus images. You can swipe both sides to see the other pictures taken.


To send the images to the AI
To add different filters on the images
To change the parameters of the picture

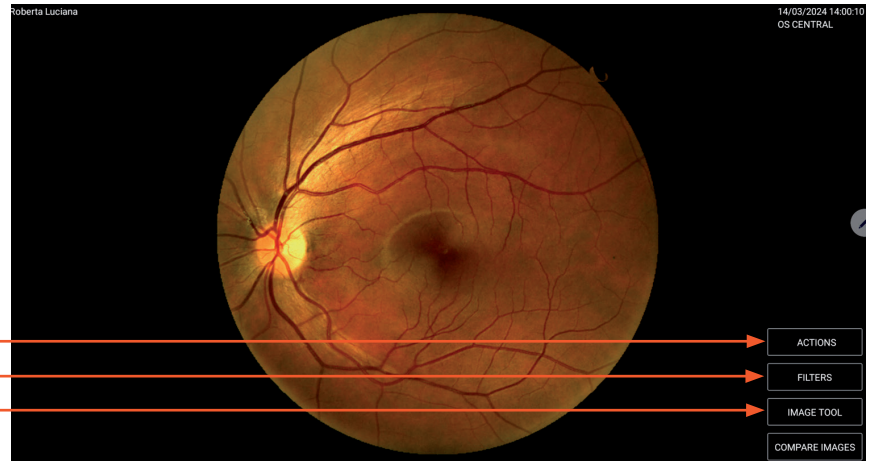

STEP 5: AI GRADING

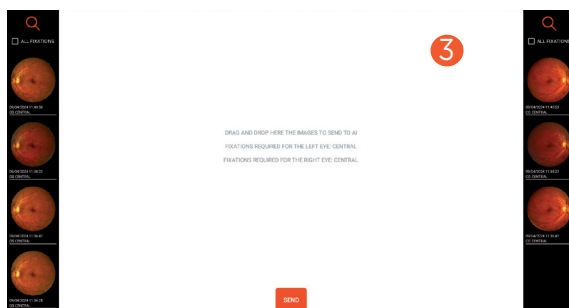

- Drag and drop the images in the center to send them to the AI. Then you can send the images.

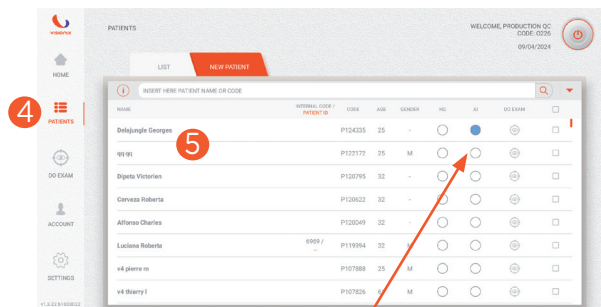

- Here, is the information that the AI report is available.
- Click on the patient's name to see the picture (6) and press the number (7) to access to the AI report.

